


Quick Guide for Indicator DP2701 (4 key)- MS21neo



Important : Before operations, make sure you read and understand the manual.


Caution : Should place the scale on the flat surface
Batteries should be kept away from children.
Only babies can be placed on the baby tray
Never leave babies unattended on the scale.

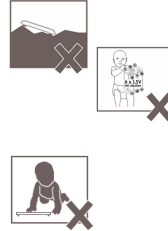


Steps :

1

YOU WILL NEED ...

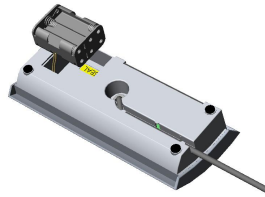

AAA x6 battery or

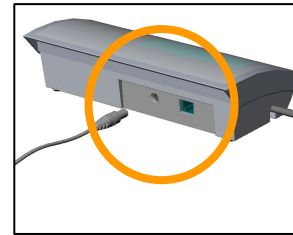

adaptor

2

Put the scale on the flat surface





and keep bubble level in the center



3

Turn on the scale.

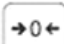Press  to zero the scale.

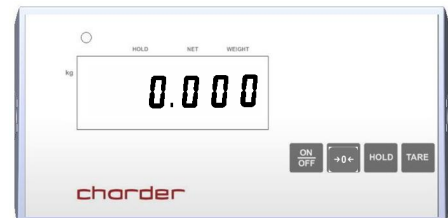


4





Press  before you place the baby on the scale.

After beep sound, step down.

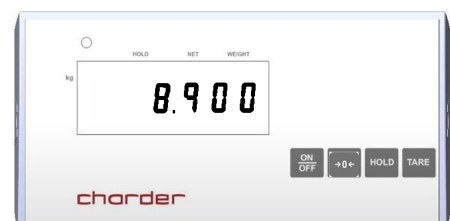Measurement is completed !!

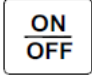

KEY FUNCTIONS (DP2701 DISPLAY- MS21neo)

Key	Description
	Turn ON/OFF the scale
	Zero the scale
	To hold weighing result
	Tare un-wanted weight



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262

TEL:+886 4 24063766

FAX:+886 4 24065612

www.chardermedical.com