


Quick Guide for model MS5400



Important : Before operations, make sure you read and understand the manual.

Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface

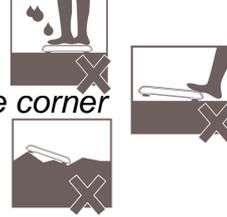

Steps :

1

YOU WILL NEED ...



AA x6 battery

2

Put the scale on the flat surface



and keep bubble level in the center



3

Turn on the scale.

Press  to zero the scale.



4

Press  before stepping on the scale.

After beep sound, step down.

Measurement is completed !!



KEY FUNCTIONS (MS5400)

Key	Description
	Turn ON/OFF the scale
	<ul style="list-style-type: none">● Determine Body Mass Index● To hold weighing result
	Tare un-wanted weight
	Zero the scale



Charder Electronic Co., Ltd

103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262

TEL:+886 4 24063766

FAX:+886 4 24065612

www.chardermedical.com