



Quick Guide for Indicator DP2701 (MS5711)

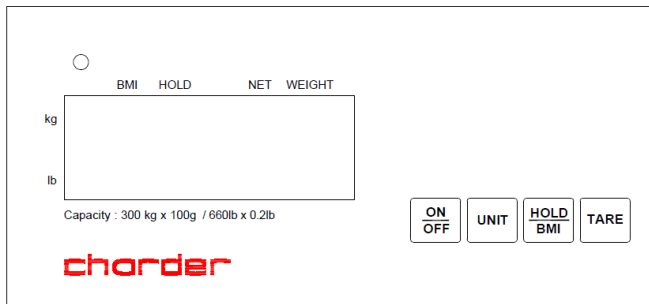

Important : Before operations, make sure you read and understand the manual.


Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface

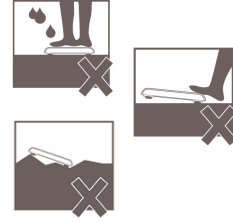


Steps :

1

YOU WILL NEED ...

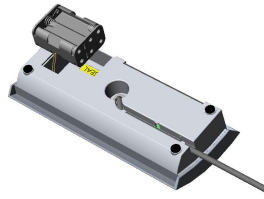

AA x4 battery or

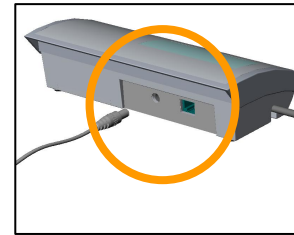

adaptor

2

Put the scale on the flat surface




and keep bubble level in the center





3

Turn on the scale.
Press ON/OFF to zero the scale.




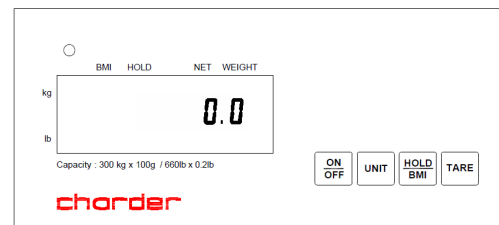


4

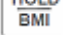Press  before stepping on the scale.
After beep sound, step down.
Measurement is completed !!

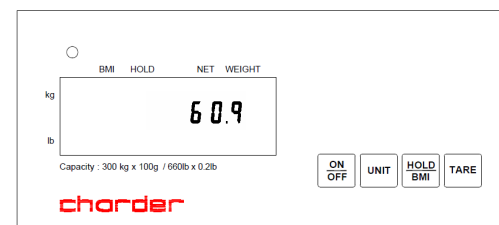

KEY FUNCTIONS (DP2701 DISPLAY- MS5711)

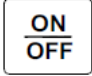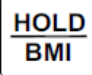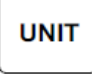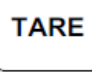Key	Description
	Turn ON/OFF the scale
	<ul style="list-style-type: none">● Determine Body Mass Index● To hold weighing result
	<ul style="list-style-type: none">● Select measuring units between kg or lb
	Tare un-wanted weight



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262
TEL:+886 4 24063766
FAX:+886 4 24065612
www.chardermedical.com